



Required tool



For your own safety
please use safe tools



For your own safety
please use safe tools



Knock out the brake cover



To be safe at cutting please hold brake as on the picture



When cutting, please cut away from you



Put replacement braking lining in hot water (80°C) for 10 min



Don't use fingers to take braking lining out



Take brake lining out of hot water with tools



Align warm brake lining with the brake wheel and push hard. Installation is easier with a heated brake lining.



Brake lining **MUST** be pushed till end of braking wheel. Situation as on picture is **NOT OK!**



To push brake lining till end of braking wheel, you can help yourself with hammer and gently pound through entire braking lining



When brake lining is installed as on picture, the installation process is complete





Not required, but we recommend that the brake lining be glued to the brake wheel at the bottom on a flat part (1 drop)



We recommend you to glue cover of braking wheel on 3 spots. (Left, right and on top side).



Push cover back on and you are DONE! 😊

Please follow installation instructions for mounting brake back on wheelchair.